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Baseline					
Collection	Manufacturer/ product number	Handling/processing in the study Center	Pipetting procedure	Storage	
Blood samples					
2x 10 ml serum tube with clotting activator without gel	BD Life Science, Franklin Lakes, USA / 367896	4 x inverted, clotting time 30-45 minutes, centrifugation 10 minutes, 15°C, 2500xg	Pipetting robot (Hamilton Robotics, Reno, USA), plates with 96 wells (Azenta Life Sciences, Chelmsford, USA) 250 µl and 500 µl tubes)	<ul style="list-style-type: none"> Central biorepository (2/3 of all collected samples): <ul style="list-style-type: none"> 500µl at -80°C 250µl tubes at -180°C Local back-up storage (1/3 of all collected samples) <ul style="list-style-type: none"> -80°C/-180°C 	
3x 10 ml K ₂ EDTA tube (1,8mg/ml)	BD Life Science, Franklin Lakes, USA / 367525	4 x inverted, 30 seconds – max. 5 minutes blood mixer, interims storage until shipping by room temperature	Pipetting robot, plates with 96 wells, separated into: <ul style="list-style-type: none"> Plasma (250 µl) Erythrocytes (250 µl tubes) Extended buffy coat (for later DNA isolation) (500 µl tubes) 	<ul style="list-style-type: none"> Central biorepository (2/3 of all collected samples) <ul style="list-style-type: none"> -180°C Local back-up storage (1/3 of all collected samples) <ul style="list-style-type: none"> -80°C/-180°C Central biorepository (2/3 of all collected samples) <ul style="list-style-type: none"> -180°C Local back-up storage (1/3 of all collected samples) <ul style="list-style-type: none"> -80°C/-180°C Central biorepository (all collected samples) <ul style="list-style-type: none"> 2x500µl -80°C, one used for DNA isolation 4x500µl -180°C 	
1x 250/400 µl gDNA	Rewity, Waltham, MA, USA / CMG-714 R	Fully automated DNA isolation with the chemagic DNA BC/Saliva Kit H96 on the chemagic Prime 8	<ul style="list-style-type: none"> DNA isolated from 1x buffy coat 	<ul style="list-style-type: none"> Central biorepository <ul style="list-style-type: none"> 1 x 250/400 µl -80°C 	
1x 175 µl, 60 ng/µl gDNA	Rewity, Waltham, MA, USA	Fully automated DNA normalisation on the chemagic Prime 8	<ul style="list-style-type: none"> Normalisation of 1x gDNA to 60 ng/µl 	<ul style="list-style-type: none"> Central biorepository <ul style="list-style-type: none"> 1 x 175 µl -80°C 	
2.5 ml RNA stabilized blood with PAXgene not yet available	BD Life Science, Franklin Lakes, USA / 762165	10 x inverted, 1-2 hours stored at room temperature, 24 hours -max. 72 hours by -20°C, then put into -80°C		<ul style="list-style-type: none"> central biorepository (all collected samples) -80°C (semi-automated storage system) Prior to storage, the RNA samples were volume reduced in central biorepository. The workflow is as follows: thaw, transfer to 50 ml flacons, centrifuge, remove supernatant, collect in resuspension buffer and vortex, transfer to Azenta tubes 	
Urine samples					

100 ml collecting cup	BD Life Science, Franklin Lakes, USA / Urine collection cup 364941, Urine tube / 365000	10 ml transferred into urine vacutainer, centrifugation 10 minutes, 15 °C, 2500xg	Pipetting robot, plates with 96 wells (250 µl and 500 µl tubes)	<ul style="list-style-type: none"> central biorepository (2/3 of all collected samples) <ul style="list-style-type: none"> 500µl at -180°C 250µl at -180°C local back-up storage (1/3 of all collected samples) <ul style="list-style-type: none"> -80°C/-180°C
Saliva samples				
Paraffin chewing gum	GC Germany, Bad Homburg, Germany,	Participant chews gum for one minute, then saliva is dispensed; if necessary, the chewing time is extended by another minute	Pipetted by hand, plates with 96 wells (500 µl tubes)	<ul style="list-style-type: none"> central biorepository (all collected samples) <ul style="list-style-type: none"> -80°C
Medicine cup with lid	Sarstedt, Nümbrecht, Germany / 75.1337.500 + 76.1340.560			
Nasal swabs				
Sterile nylon flock swabs	Copan Flock Technologies, Brescia, Italy	Two swabs are transferred in one 2 ml cryotube pre-filled with RNA-later	-	<ul style="list-style-type: none"> central biorepository (all collected samples) <ul style="list-style-type: none"> -80°C
Stool samples				
At the same day or one day before the visit in the study center, at home with stool catcher for sample collection	Süsse Labortechnik, Gudensberg, Germany / S1000	Stabilized stool sample is sent to the central university laboratory Greifswald, where it is aliquoted in 500 µl tubes	-	<ul style="list-style-type: none"> central biorepository (all collected samples) <ul style="list-style-type: none"> -80°C
Two stool collecting tubes were filled:	Biosepar, Mühldorf am Inn, Germany			
<ul style="list-style-type: none"> one is without additives 				
<ul style="list-style-type: none"> one tube was prefilled with 3.5ml RNA-later (RNA stabilizing additive with K2EDTA) 	Quiagen, Hilden, Germany			
The samples are packed in a protective bag "Süsse Schutzbeutel" in a box "Süsse Post Box	Süsse Labortechnik, Gudensberg, Germany / H64490 and H11227			

Slim" and brought to the study center at the time of the examination					
Follow-up examination					
Collection	Manufacturer/ product number	product	Handling/processing in the study Center	Pipetting procedure	Storage
Blood samples					
1x 10 ml serum tube with clotting activator without gel	BD Life Science, Franklin Lakes, USA / 367896		4 x inverted, clotting time 30-45 minutes, centrifugation 10 minutes, 15°C, 2500xg	Pipetting robot (Hamilton Robotics, Reno, USA), plates with 96 wells (Azena Life Sciences, Chelmsford, USA) 250 µl tubes)	<ul style="list-style-type: none"> • Central biorepository (2/3 of all collected samples): <ul style="list-style-type: none"> ○ 1x250µl tube at -80°C* ○ 7x250µl tubes at -180°C • Local back-up storage (1/3 of all collected samples) <ul style="list-style-type: none"> ○ -80°C/-180°C
2x 10 ml K ₂ EDTA tube (1,8mg/ml)	BD Life Science, Franklin Lakes, USA / 367525		4 x inverted, 30 seconds – max. 5 minutes blood mixer, interims storage until shipping by room temperature	Pipetting robot, plates with 96 wells, separated into: <ul style="list-style-type: none"> • Plasma (250 µl tubes) • Extended buffy coat (for later DNA isolation) (500 µl tubes) 	<ul style="list-style-type: none"> • Central biorepository (2/3 of all collected samples) <ul style="list-style-type: none"> ○ 250 µl at -180°C • Local back-up storage (1/3 of all collected samples) <ul style="list-style-type: none"> ○ -80°C/-180°C • Central biorepository (2/3 of all collected samples) <ul style="list-style-type: none"> ○ 1x500µl -80°C** ○ 1x500µl -180°C • Local back-up storage (1/3 of all collected samples) <ul style="list-style-type: none"> ○ -80°C/-180°C
Urine samples					
100 ml collecting cup	BD Life Science, Franklin Lakes, USA / Urine collection cup 364941, Urine tube / 365000		10 ml transferred into urine vacutainer, centrifugation 10 minutes, 15 °C, 2500xg	Pipetting robot, plates with 96 wells (250 µl)	<ul style="list-style-type: none"> • central biorepository (2/3 of all collected samples) <ul style="list-style-type: none"> ○ 250µl at -180°C • local back-up storage (1/3 of all collected samples) <ul style="list-style-type: none"> ○ -80°C/-180°C
Second Follow-Up					
Collection	Manufacturer/ product number	product	Handling/processing in the study Center	Pipetting procedure	Storage
Blood samples					
1x 10 ml serum tube with clotting activator without gel	BD Life Science, Franklin Lakes, USA / 367896		4 x inverted, clotting time 30-45 minutes, centrifugation 10 minutes, 15°C, 2500xg	Pipetting robot (Hamilton Robotics, Reno, USA), plates with 96 wells (Azena Life Sciences, Chelmsford, USA) 250 µl and 500 µl tubes)	<ul style="list-style-type: none"> • Central biorepository (2/3 of all collected samples): <ul style="list-style-type: none"> ○ 500µl at -80°C ○ 250µl tubes at -180°C • Local back-up storage (1/3 of all collected samples) <ul style="list-style-type: none"> ○ -80°C/-180°C

2x 10 ml K ₂ EDTA tube (1,8mg/ml)	BD Life Science, Franklin Lakes, USA / 367525	4 x inverted, 30 seconds – max. 5 minutes blood mixer, interims storage until shipping by room temperature	Pipetting robot, plates with 96 wells, separated into: <ul style="list-style-type: none"> Plasma (250 µl) Extended buffy coat (for later DNA isolation) (500 µl tubes) 	<ul style="list-style-type: none"> Central biorepository (2/3 of all collected samples) <ul style="list-style-type: none"> -180°C Local back-up storage (1/3 of all collected samples) <ul style="list-style-type: none"> -80°C/-180°C Central biorepository (all collected samples) <ul style="list-style-type: none"> 2x500µl -80°C, one used for DNA isolation 4x500µl -180°C
1x 10 ml lithium-heparin tube	BD Life Science, Franklin Lakes, USA / 367526	4 x inverted, 30 seconds – max. 5 minutes blood mixer, interims storage until shipping by room temperature	Pipetting robot, plates with 96 wells, separated into: <ul style="list-style-type: none"> Plasma (250 µl) 	<ul style="list-style-type: none"> Central biorepository (2/3 of all collected samples) <ul style="list-style-type: none"> -180°C Local back-up storage (1/3 of all collected samples) <ul style="list-style-type: none"> -80°C/-180°C
2.5 ml RNA stabilized blood with PAXgene not yet available	BD Life Science, Franklin Lakes, USA / 762165	10 x inverted, 1-2 hours stored at room temperature, 24 hours -max. 72 hours by -20°C, then put into -80°C		<ul style="list-style-type: none"> central biorepository (all collected samples) -80°C (semi-automated storage system) Prior to storage, the RNA samples were volume reduced in central biorepository. The workflow is as follows: thaw, transfer to 50 ml flacons, centrifuge, remove supernatant, collect in resuspension buffer and vortex, transfer to Azenta tubes
Urine samples				
100 ml collecting cup	BD Life Science, Franklin Lakes, USA / Urine collection cup 364941, Urine tube / 365000	10 ml transferred into urine vacutainer, centrifugation 10 minutes, 15 °C, 2500xg	Pipetting robot, plates with 96 wells (250 µl and 500 µl tubes)	<ul style="list-style-type: none"> central biorepository (2/3 of all collected samples) <ul style="list-style-type: none"> 500µl at -180°C 250µl at -180°C local back-up storage (1/3 of all collected samples) <ul style="list-style-type: none"> -80°C/-180°C
Stool samples				
At the same day or one day before the visit in the study center, at home with stool catcher for sample collection	Süsse Labortechnik, Gudensberg, Germany / S1000	Stabilized stool sample is sent to the central university laboratory Greifswald, where it is aliquoted in 500 µl tubes	-	<ul style="list-style-type: none"> central biorepository (all collected samples) <ul style="list-style-type: none"> -80°C
Stool collecting tube was filled:	Biosepar, Mühldorf am Inn, Germany			

<ul style="list-style-type: none">one tube was prefilled with 3.5ml RNA-later (RNA stabilizing additive with K2EDTA)	Quiagen, Hilden, Germany
The samples are packed in a protective bag "Süsse Schutzbeutel" in a box "Süsse Post Box Slim" and brought to the study center at the time of the examination	Süsse Labortechnik, Gudensberg, Germany / H64490 and H11227

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Sample type	Volume	Aliquots per subject (Total/in Biorep)	Study participants (Completeness, %)	Study participants at central storage (Completeness, %)	Retrievable	Preprocessing required	Preferred analysis
Baseline examination*							
Plasma	250 µl	48/32	97.1	94.9	Yes (-180°C storage temperature)	No	
Serum	250 µl	30/20	97.4	96.0	Yes (-180°C storage temperature)	No	
Serum	500 µl	1/1	96.3	94.3	Yes (-180°C storage temperature)	No	
Erythrocytes	250 µl	6/4	96.0	94.6	Yes (-180°C storage temperature)	No	
Buffy Coat	500 µl	6/5	95.5	93.7	Yes (-80°C and -180°C storage temperature)	DNA isolation	DNA isolation → genomics
gDNA	400 (250) µl	1/1	88.3	88.3	Yes (-80°C storage temperature)		
gDNA 60 ng/µl Normalisation	175 µl	1/1			Yes (-80°C storage temperature)	Currently underway at the HMGU	Genotyping, Whole Genome Sequencing, any genomic analysis
Urine	250 µl	12/8	97.3	95.9	Yes (-180°C storage temperature)	No	
Urine	500 µl	4/4	96.4	94.6	Yes (-180°C storage temperature)	No	
Saliva	500 µl	2/2	93.5	92.5	Yes (-80°C storage temperature)	DNA isolation, to be done by user	DNA isolation → genomics (microbiome)
Nasal Swab	2 ml	1/1	93,5	91,5	Yes (-80°C storage temperature)	DNA isolation, to be done by user	DNA isolation → genomics (microbiome)
Native Stool	5 ml	1/1	78.3	75.3	Yes (-80°C storage temperature)	No	
Stabilised Stool	500 µl	4/4	84.2	78.3	Yes (-80°C storage temperature)	RNA isolation	RNA isolation → transcriptomics
RNA (PAX Gene) not yet available	10 ml	1/1	9.9	7.9	No, will be volume reduced (500 µl) at the central Biorepository	-	-
RNA (500 µl) not yet available	500 µl	1/1	83.6	83.5	Yes, depending on the state of the RNA volume reduction (-80°C storage temperature)	RNA isolation, to be done by user	RNA isolation → transcriptomics
Follow-up examination**							

Plasma	250 µl	18/12	97.9	94,6	Yes (-180°C storage temperature)	No	
Serum	250 µl	12/8	97.2	94.7	Yes (-80°C samples since August 2022 and -180°C storage temperature)	No	
Buffy Coat	500 µl	2/2	96.8	88,8	Yes (-180°C storage temperature)	DNA isolation, extraction will take place at HMGU, stored at -80 C	DNA isolation → genomics
Urine	250 µl	6/4	98.7	96.1	Yes (-180°C storage temperature)	No	
Second Follow-up examination ***							
Buffy Coat	500 µl	3/2	12.6	7.3	Yes (-80°C storage temperature)	DNA isolation, extraction will take place at HMGU, stored at -80 C	DNA isolation → genomics
Plasma	250 µl	18/12	12.8	7.2	Yes (-180°C storage temperature)	No	
Heparin-Plasma	250 µl	10/6	12.6	7.1	Yes (-180°C storage temperature)	No	
RNA (PAX Gene) not yet available	10ml	1/1	12.1	4.1	Yes (-80°C storage temperature)	-	
RNA (500 µl) not yet available	500 µl	1/1	0	0	Yes, depending on the state of the RNA volume reduction (-80°C storage temperature)	RNA isolation, to be done by user	RNA isolation → transcriptomics
Serum	250 µl	12/8	12.6	7.2	Yes (-180°C storage temperature)	No	
Serum	500 µl	1/1	12.5	7.1	Yes (-180°C storage temperature)	No	
Urine	250 µl	6/4	12.8	7.4	Yes (-180°C storage temperature)	No	
Urine	500 µl	6/4	12.8	7.5	Yes (-180°C storage temperature)	No	

Numbers are subject to change when samples are released from stock or consent is withdrawn.

*Planned number of participants for baseline examination has been set to 205,000 except for stool samples. Collection of stool samples had not been introduced by the start of the study, but only in October 2016. In the case of the stool samples, the planned number of participants was set at 100,000.

**Planned number of participants for follow-up examination has been set to 135,000.

*** Planned number of participants for second follow-up examination has been set 87.000

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